Prothesoft®

SOFT & SECURE DENTURE LINING

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For over 20 years, Prothesoft® has been synonymous with safe, efficient, and patient-friendly relining of full dentures. With the MDR-compliant reintroduction of Prothesoft® Forte, we offer dental laboratories and dentists a patented, permanently soft material that is highly comfortable to wear and offers cost advantages for patients.

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The problem: jaw atrophy and insufficient support

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Prothesoft® is a patented material for permanently soft relining of full dentures.

- *Processed in the dental laboratory*
- *Adapts dynamically to the jaw*
- *Forms an elastic intermediate layer that absorbs stress peaks*
- *Ensures a secure fit, high wearing comfort and long-lasting functionality of the denture*

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- *Over 20 years of clinical experience*
- *No known side effects or risks*
- *MDR-compliant and CE-marked (Class I)*
- *Shelf life: 2 years when stored in a dry, dark place below 15 °C*
- *Documented patient satisfaction and high laboratory safety in processing*

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Prothesoft® Forte: 4-step processing

1

Preparation of the denture: After taking impressions and, if necessary, functional impressions, grind down the inside of the denture by 1–2 mm and degrease it.

Preparation of the gel: Prothesoft® Forte consists of bottle A (20 g liquid base substance) and bottle B (3.4 g powdered peroxide). The powder is added completely to the liquid, the mixture is shaken vigorously and then heated for 2 minutes in a water bath (90–100 °C). This produces a viscous gel.

2

3

Polymerisation: The gel is applied evenly, the cuvette is pressed under 2–3 bar and polymerised for two hours in a water bath at 90–100 °C.

Finishing and polishing: After cooling, the denture is removed and air-dried for 15–30 minutes. It is then finished using conventional tools.

4

Recommendation: Before integration, it is advisable to store the material in a water bath in order to flush out any remaining monomers and accelerate the swelling process.

Storage: below 15 °C, dry and dark
Shelf life: 24 months from date of manufacture
Application: exclusively in dental laboratories

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